# HOW DO I TAKE THE STOOL TEST?

## FOLLOW THESE USER INSTRUCTIONS STEP BY STEP.

To see a video, visit

[www.dikkedarmkanker.bevolkingsonderzoek.be/en/ddk/enghow-examination-conducted](http://www.dikkedarmkanker.bevolkingsonderzoek.be/en/ddk/enghow-examination-conducted).

**IMPORTANT:** ▶ Use the test before the expiry date on the vial: 2022/04. ▶ Store the stool test at room temperature before you use it. ▶ Please send the padded envelope containing your sample as soon as possible. Do not send your sample on a Friday or the day before a holiday so that it is not too long in the postbox. ▶ Defer participation at temperatures above 25°C. ▶ Place the stool test in a prominent place on the toilet so that you do not forget it!

**STEP 1:** Open the padded envelope and remove the contents: ▶ a pouch containing stool test and absorption paper (rectangular white tissue) ▶ collection paper (white paper to collect stool)

**STEP 2:** Go to the toilet. Urinate and flush. Follow the instructions on the collection paper and attach it to the toilet seat. **Note: The collection paper must not come into contact with water or urine.**

**STEP 3:** Your stool is collected at the centre of the collection paper. Remove the vial from the clear pouch, leaving the white tissue in place. Twist the green cap (open here) and open the vial.

**STEP 4:** There is a probe with a ribbed tip on the green cap. Pierce your stool in four different places. Only a very small amount of stool needs to be attached to the ridged tip (figure 5).

**STEP 5:** Insert the green cap with the probe into the vial. Tighten it securely. **Note: Do not write anything on the vial.**

**STEP 6:** Place the vial in the pouch, leaving the white tissue in place. Close the pouch. Detach the collection paper and drop it into the toilet. Let the paper soften in the water. Flush the collection paper containing your stool.

**STEP 7:** Complete the participation form. Check your GP/doctor's details and amend, if necessary. **Note: Without a correctly completed participation form, you will not receive the results!**

**STEP 8:** Place the pouch and the participation form together in the padded envelope. The address of the lab is on this envelope.

**STEP 9:** Please send it as soon as possible. A postage stamp is not required. Are you not able to send the stool test straight away? Store it in the refrigerator for up to 14 days.

# WHAT HAPPENS TO MY SAMPLE?

The lab will test your stool sample for the presence of blood. You and your GP/doctor will receive the result within 14 days. This will always be given, even when everything is alright.

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FOB Gold® is a trademark in various jurisdictions, which is exclusively licensed to SENTINEL CH. SpA.
■ Sentinel CH. SpA Via Robert Koch 2, 20152 Milan, Italy www.sentinel-diagnostics.com